



INTENDED USE:

Cleaner is a stabilized and micro-filtered concentrated hypochlorite reagent for intensive oxidative cleaning, rinsing and washing of hematology analyzers' capillaries, tubing, chambers, removing blood component precipitates and lipoprotein deposits in 3-part Hematology analyzers.

Please refer to the analyzer's User Manual for further information.

ACTIVE INGREDIENTS:

Sodium Hydroxide	: 0.5 %
Sodium Hypochlorite solution	: 5.0 %
Stabilizers	: 0.5 %

DIRECTION FOR USE:

1. Personnel installing the reagent, should be a trained laboratory professional.
2. Open the container cover and use the Cleaner as per instructions given in the analyzer's User Manual.
3. Avoid contamination of the tubing and the reagent.
4. Do not pour and mix the remains of a reagent from a container, into another one.
5. Prime thoroughly with the new reagent and measure background values according to the analyzer's User Manual.
6. When installing a new lot of the reagent, recalibrate the instrument as specified in the User Manual.

PRECAUTIONS:

- a. Cleaner, is for *in vitro* diagnostic use only.
- b. Cleaner is environmental friendly and does not contain harmful ingredients.
- c. Cleaner contains sodium hypochlorite in strongly alkaline solution.
- d. Do not inhale the reagent or pipette by mouth.
- e. Avoid contact with skin/eyes. In case of contact, wash off immediately with plenty of water and seek medical attention as early as possible.
- f. Do not use if package is damaged.
- g. Keep the reagent container closed when not in use.
- h. Wear protective gloves when handling the reagents.
- i. Please refer to the MSDS associated with the reagents.
- j. Use Good Laboratory Practices (GLP) when handling the reagents.

SPECIMEN COLLECTION AND PREPARATION:

Blood specimens collected by vein puncture in EDTA anticoagulant. Specimens for hematological analysis may be stored at 15 – 30°C for 2 – 4 hours or at 2 – 8°C for 24 hours after collection.

REAGENT PREPARATION:

This reagent is ready for use and can be applied straight from the container; no special reagent preparation is necessary. This reagent does not affect the number of tests performed. This is used for the cleaning and maintenance of the instrument.

STORAGE, STABILITY AND DISPOSAL:

- Store Cleaner between 15 and 30°C.
- The unopened reagent is stable up to the expiry date mentioned on the label when stored between 15°C – 30°C.
- Use Cleaner within 06 months of first opening.
- Do not use reagent beyond the expiration date printed on the container label.
- Discard opened container after 06 months.
- Do not use reagent once frozen.
- Dispose waste product in accordance with local Govt. regulations.

Manufactured by:

Vanguard Diagnostics Private Limited
C - 123, Phase – 1, Okhla Industrial Area,
New Delhi – 110020

Phone: +91-11-41727222, +91-11-41084222
Email: info@vanguarddiagnostics.com
Website: www.vanguarddiagnostics.com